


From Serial number 5100.20.011

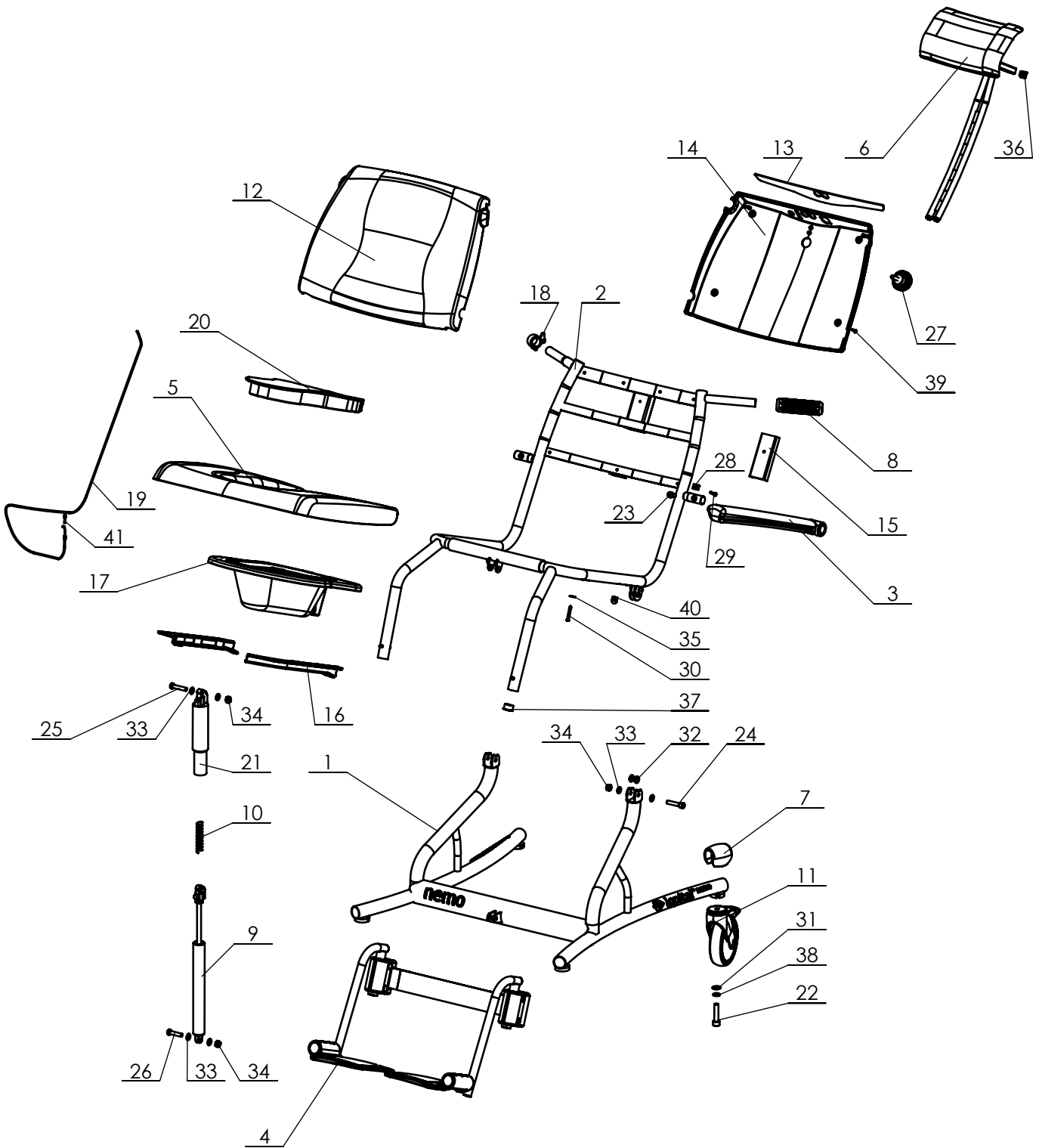

C:\Backup\_16-05-2020\SW-Tek\Nemo 5100\

Signed	RH	Date:	19-01-2018
Revised:	01 / KA	Date:	26-10-2018
Revised:	02 / KA	Date:	03-04-2020
Copyright reserved by law		Unit of measurement: mm	

# Overview parts Nemo

Lopital Nederland bv

Type nr: 51005100

## Nemo 51005100

Part nr.	QTY.	Art.nr.	Part
1	1	70510001	Chassis
2	1	70510002	Seat frame
3	2	70565001	Armrest
4	1	69605199	Footrest complete
5	1	70560046	Seat
6	1	68405630	Head/neck support complete
7	4	70560066	Protective cover wheel
8	2	70560049	Handgrip
9	1	70510003	Gasspring
10	1	70510013	Compression spring
11	4	70519928	Castor Ø 125 mm braked
12	1	70560047	Backrest
13	1	70510010	Sticker Rugkap
14	1	70560053	Backrest cover
15	1	70560039	Slide block
16	1	70510050	Guide for bedpan set
17	1	69501104	Bedpan + lid
18	1	70510008	Gasspring hendel
19	1	70510004	Gasspring cable
20	1	70560045	Lid for seat
21	1	70510009	Cover gasspring
22	4	70519935	Cylinder head screw M10 x 45
23	2	75005640	Cylinder head screw M8x16 Super Duplex 4529
24	2		Cylinder head screw M8x40
25	1		Cylinder low head screw M8x40
26	1		Cylinder low head screw M8x35
27	1		Rotary knob M8
28	4		Snap nut M4
29	7		Chipboard screw 4x20
30	4		Chipboard screw 5x35
31	4		Washer M10
32	4		Flanged Slide bearing 8x12x6
33	8		Washer M8
34	4		Locknut M8
35	4		Washer M5 Ø15
36	1		Tube cap 15x15
37	2		Tube cap Ø25
38	4		Nord-Lock M10
39	4		Chipboard screw 3,9x16
40	1		Tube cap Ø18 dome
41	1		Shrink tube

Signed	RH	Date:	19-01-2018
Revised:	<b>01 / KA</b>	Date:	26-10-2018
Revised:	<b>02 / KA</b>	Date:	03-04-2020

## Overview parts Nemo

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 51005100



C:\Backup 16-03-2020, SW-Tek\Nemo 5100\

Signed	RH	Date:	19-01-2018
Revised:	<b>01 / KA</b>	Date:	26-10-2018
Revised:	<b>02 / KA</b>	Date:	03-04-2020

## Overview parts Nemo

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 51005100